


FUSION SLIDER SET UP INSTRUCTIONS

Needed Parts:


- 1 kit Lid
- 1 set of Tiltfeet
- 1 Fusion Slider
- Anaglyphs
- R/G Clip
- Mainframe

Instructions:


1. Place the shorter side of the kit lid into the vertical slots in the tilt feet.

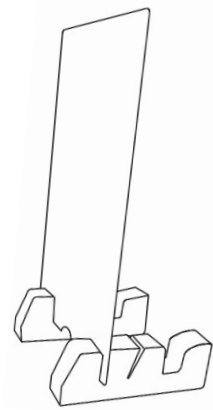


2. Take the fusion slider ruler with the following orientation.

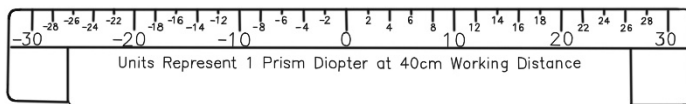


3. Place the fusion ruler over the top of the kit lid and slide down just under halfway.

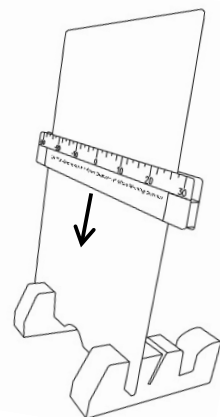

4. Set the anaglyphs on the front of the lid behind the ruler, with the red anaglyph in front of the green anaglyph.

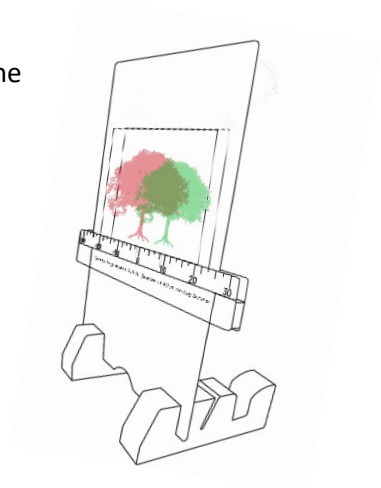

5. Attach the R/G clip to the mainframe with red lens over the right eye.

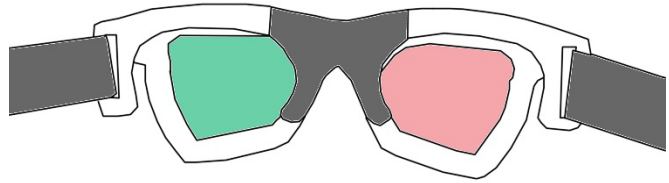

*If you are having a difficult time viewing the anaglyph because the background is too busy, try placing the lid in the angled slot of the tiltfeet.