


Fusion Slider Lite Instructions

The Fusion Slider Lite is designed as a way to hold fusion cards or anaglyphs with your hand or to place on a desk.


General Instructions:

The slider can bend in and out to accommodate 2 overlapping anaglyphs or a single fusion card. Simply open the slider, place in the desired card or anaglyphs and close it tightly

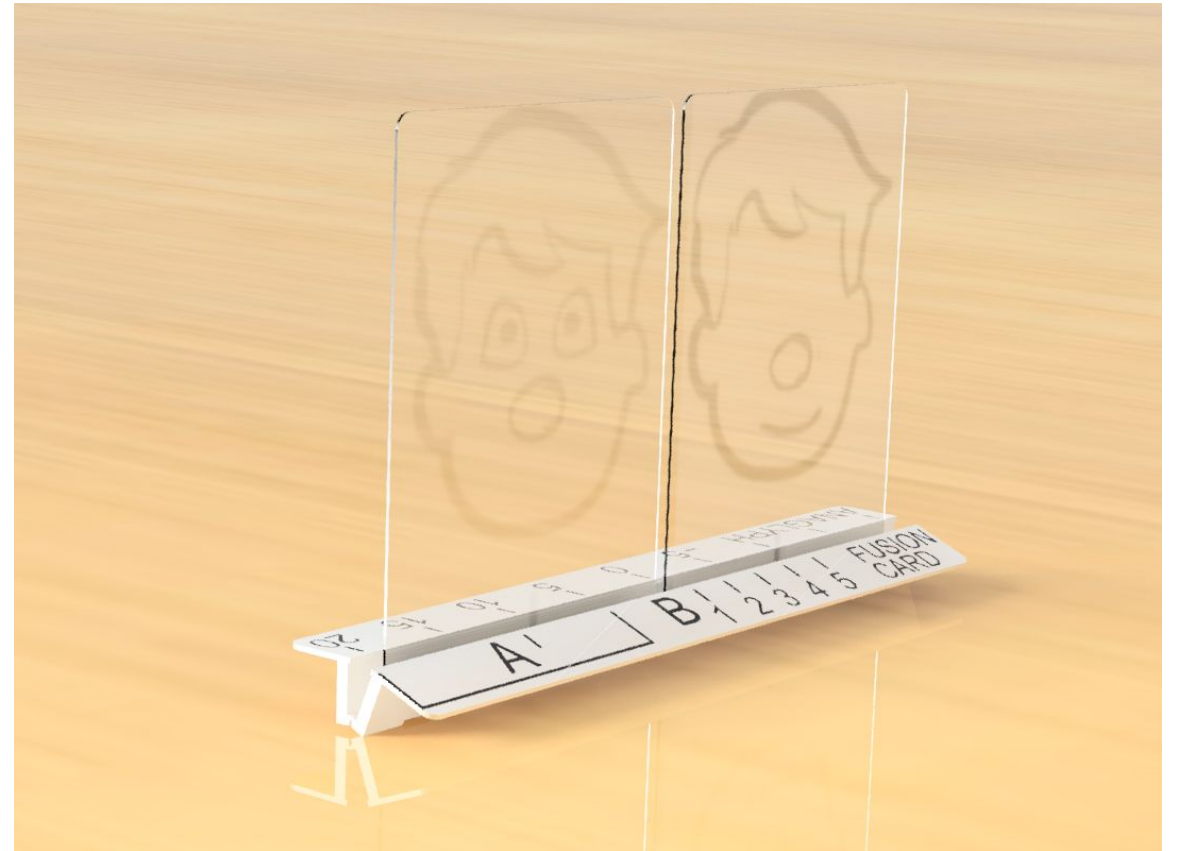

Fusion Cards

Peel the blue protective coating from the Transparent Cards

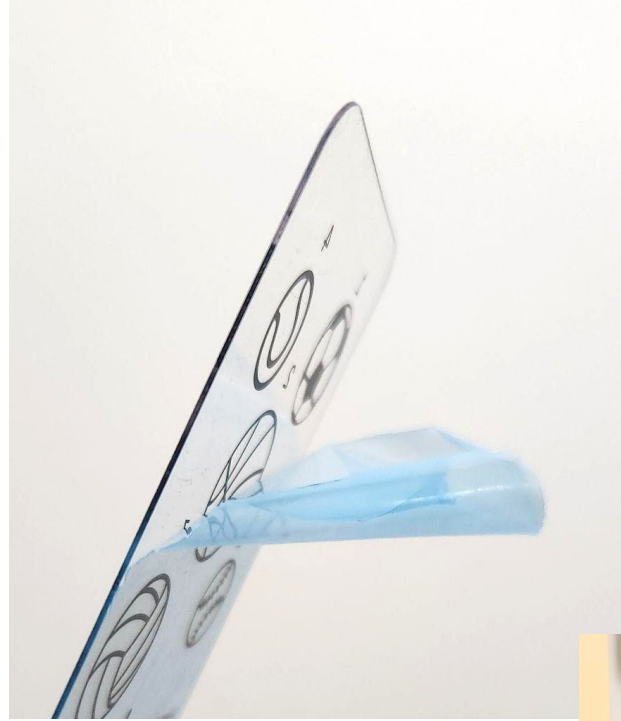


Place Card A in the slider as indicated, lining up the hash mark on the card with the hash mark on the slider inside of bracket A.




Align Card B's hashmark with the hashmarks on the slider labeled 1 through 5 depending on the instructions of your vision therapist







Hold the fusion card in front of your eyes about 16 in from your eyes or as instructed by your vision therapist

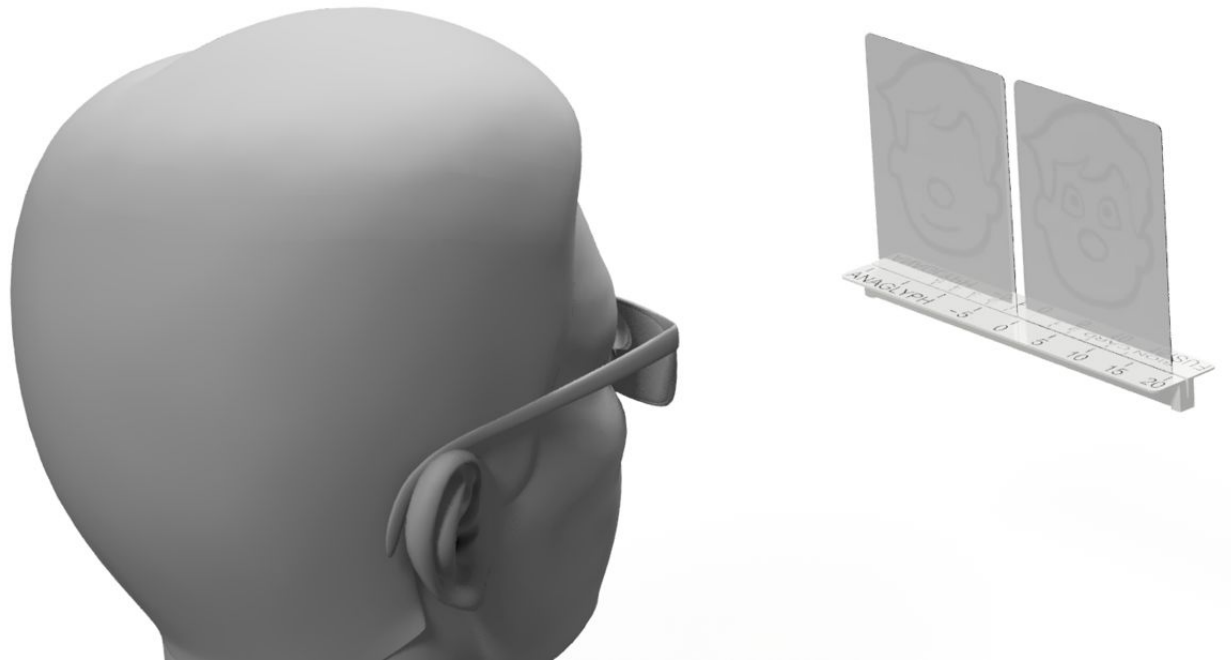

Your therapist may use a binder clip to secure the fusion card at a certain location.



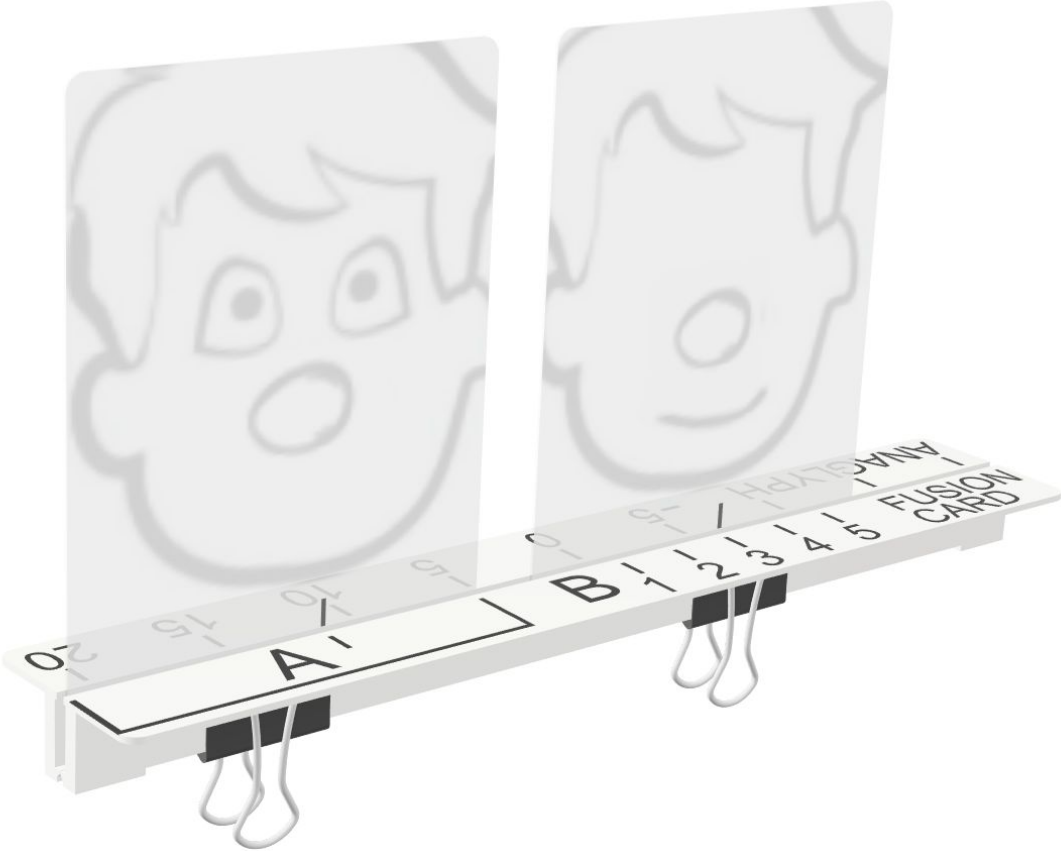


The slider can be held or placed on a desk with or without the binder clips

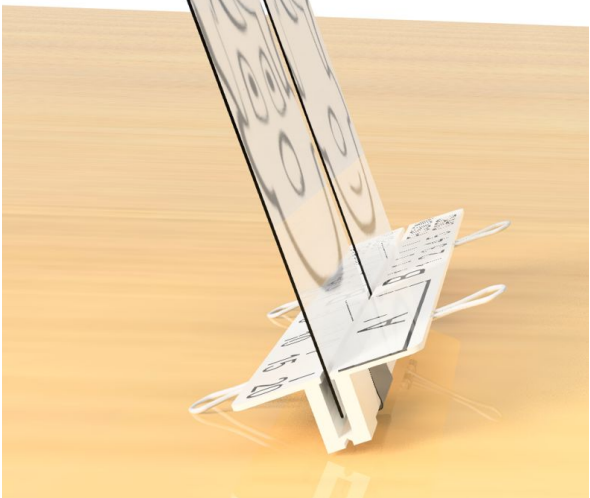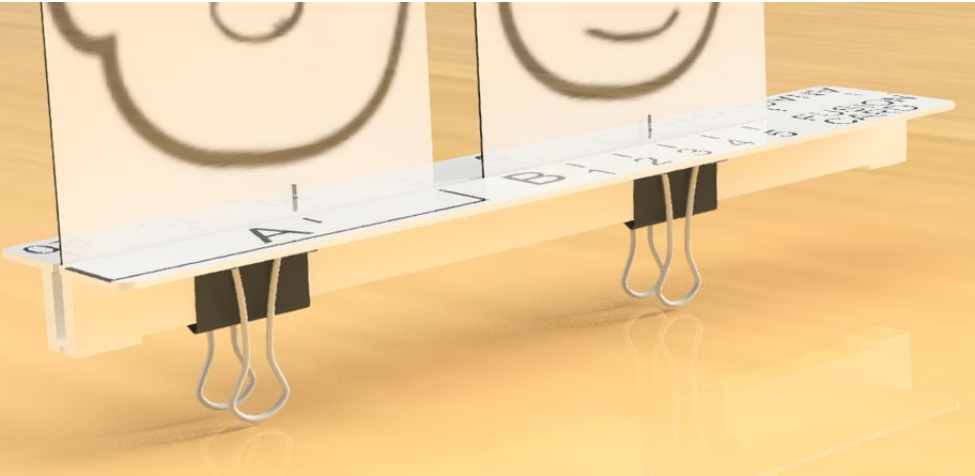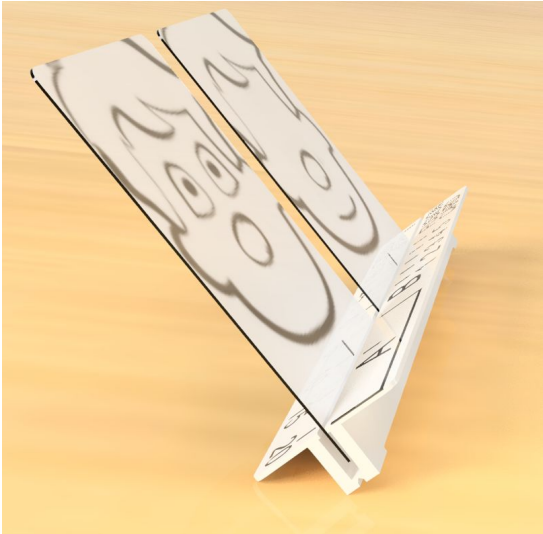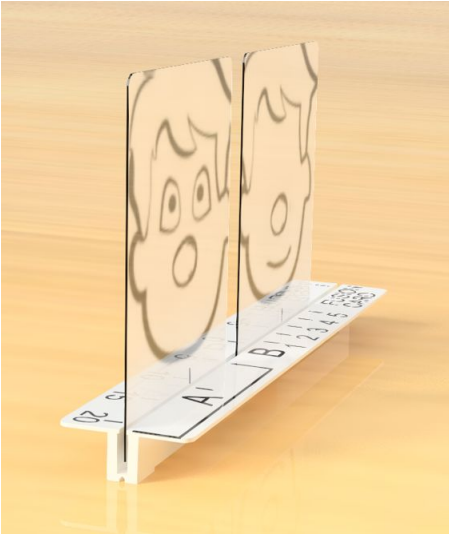


Anaglyphs


1. Place the Magnetic Forehead Mount (coming soon - use magnetic main frame for now) on and place the R/G Magnetic Clip (glasses) on with the **red lens over the right eye**.
2. Place the pair of anaglyphs in the holder with the **Red over the Green** anaglyph

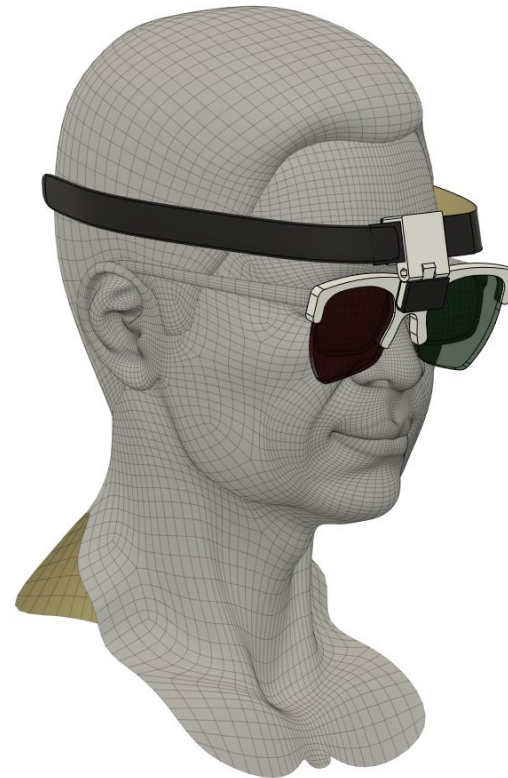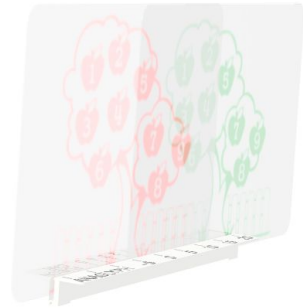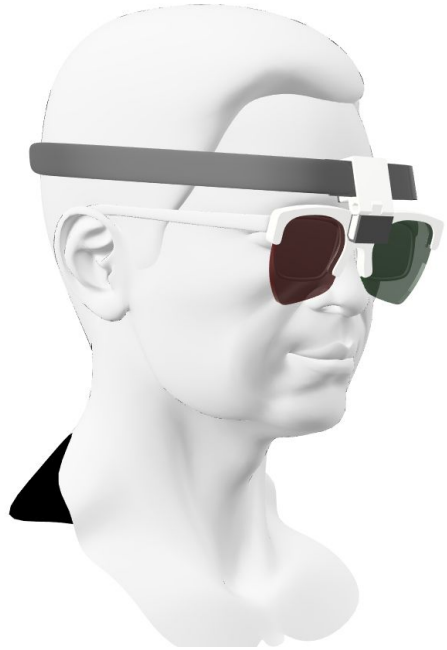

Place the Green Anaglyph center hashmark at zero

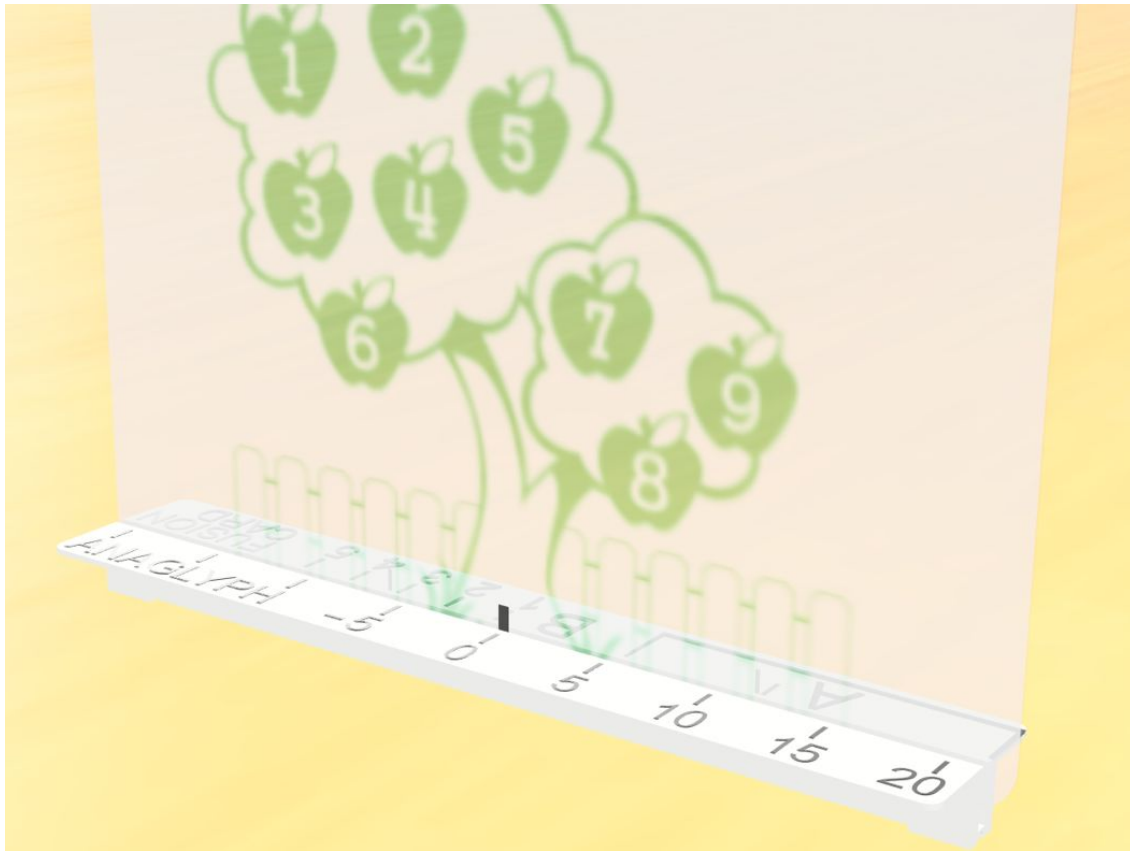

Place the Red Anaglyph center hash mark at the hashmark instructed by your therapist.

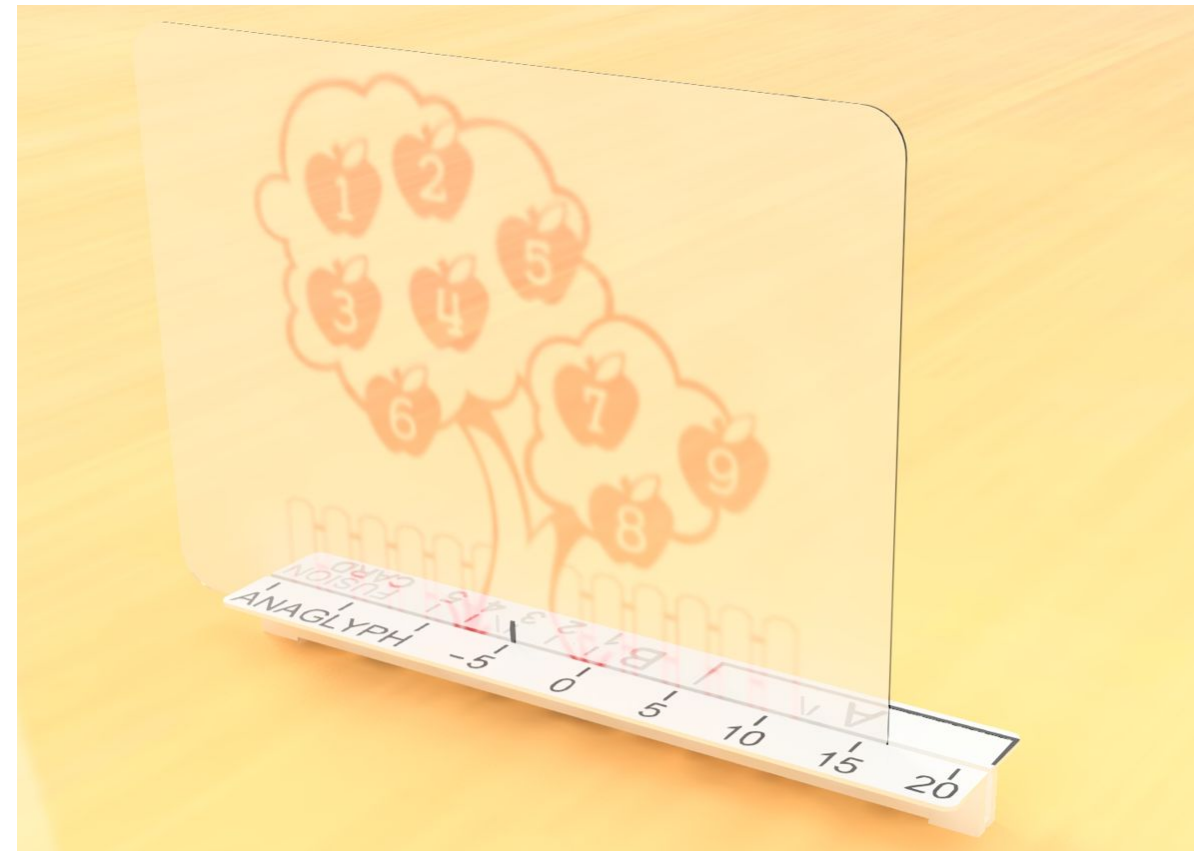

Your therapist may use a binder clip to secure the anaglyphs at a certain location.



The slider can be held or placed on a desk with or without the binder clips

