


Instructions for the Magnetic Forehead Mount

The **Magnetic Forehead Mount** is designed for comfort and ease of use, allowing you to attach various magnetic accessories, including the **Emergent Red/Green Clip (glasses)**, **Magnetic Occluder**, **Magnetic Red Lens**, and **Premium Magnetic Bamboo Flippers**, as directed by your vision therapist.


How to Wear the Magnetic Forehead Mount:

1. **Position It** – Place the black padded side against the center of your forehead, just above your eyebrows.
2. **Secure It** – Pull the strap over your head and tighten the Velcro until it feels snug but comfortable.

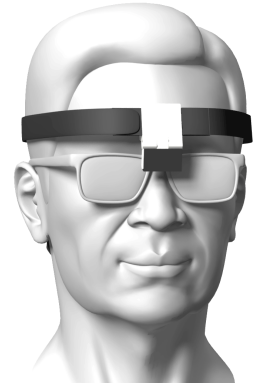

Attaching Magnetic Accessories:

- Normally, attach magnetic accessories **BEHIND** the magnetic padded bracket, but they can also go in front.
- For the **Emergent Red/Green Clip and Occluder**, place it **behind the bracket** for optimal positioning close to your eyes, touching the cheek bone, or as instructed by your vision therapist.
- When using **Lens Flippers with an Occluder or Red/Green Clip**, place the **occluder or clip behind the bracket** and the **Lens Flipper in front**.

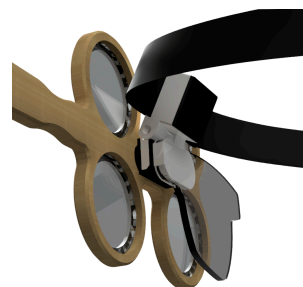

Adjusting for Use With or Without Glasses:

- **With Glasses** – Adjust the magnetic padded bracket so it sits just in front of the bridge of your glasses. You may need to remove the mount to reposition it.
- **Without Glasses** – Tilt the magnetic padded bracket slightly toward your cheekbones for a better fit. Removing the mount may help with adjustments.